

# Aurora CL<sup>2</sup> & Alliance CL<sup>2</sup>

## To Set-up the Table:



Fig. 1

- Place the folded table sitting on the rubber bumpers; with the heel of your hand, bump the tab between the face rest outlets and the table will open. (Fig. 1)
- Open the table up until the legs are fully extended. Remove the face rest. (Fig. 2)



Fig. 2

## To Set the Table Upright:

- When the table is fully opened, tilt the table upright until all four feet are on the floor. Lift one end of the table, press down on the middle and tension the cables a little. (Fig. 3)



Fig. 3

## To Change the Table Height:

- Choose the hole that represents the desired height and after removing the knob, put the bolt through that hole. Replace and tighten the knob by hand until snug. Repeat on all other leg extensions, setting all extensions at same height. (Fig. 4)



Fig. 4

## To Open/Adjust/Close Tilt Backrest Section on Alliance:



Fig. 5

- To Open/Adjust, release velcro pull tab located under the center at the end of the table and pull tab. Pull up on the center of the backrest, stopping at desired height (Fig. 5).
- Make sure that both sides of the tilting hardware are on the same setting and are both locked in place.
- To close, raise the backrest fully and lower to closed position. Re-attach velcro tab to hold backrest in closed position. Refer to warning label on underside of table for precautions.

## To Close the Table:



Fig. 6

- Turn the table on its side with the rubber bumpers down. If you own a face-rest, store it under the table by pushing the insertion dowel down into the clip provided. (Fig. 6) Partially close the table top.
- Fold the legs into the table and make sure the armrest is laying flat and the cables are inside of the table. Push the two halves of your table together to lock. (Fig. 7).



Fig. 7

**CAUTION**

Dynamic Load Rating:

**400 lbs.**

**Handcrafted in the U.S.A.**

Inspected By: \_\_\_\_\_

Oakworks®  
Original



OAKWORKS®, Inc.

923 East Wellspring Rd., New Freedom, PA 17349

All portable tables: U.S. Patent 5,009,170  
Unilock Patent Pending

Toll Free Phone: 800-558-8850 Phone: 717-235-6807  
Toll Free FAX: 877-562-4787 FAX: 717-235-6798

www.oakworks.com

