


instruction sheet

READY RAMP weight cap. 600 lbs.

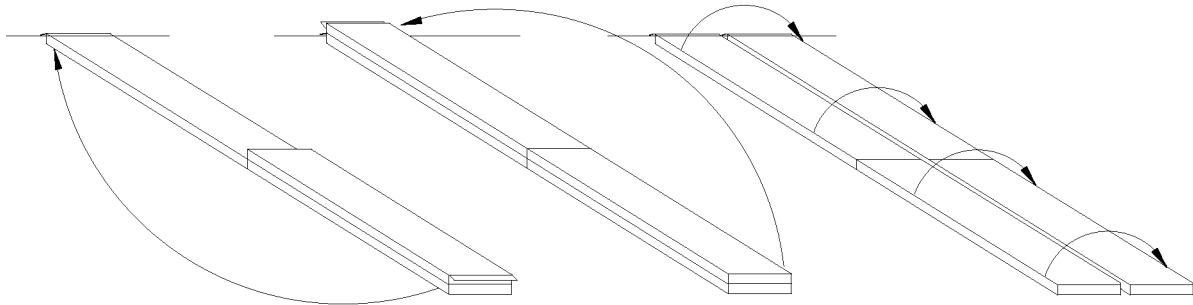


Instructions

STEP 1. Unfold the First Section. Position the top adapter in the door or landing as shown. Be sure that the complete top adapter lip of the ramp is securely placed on the landing surface.

STEP 2. Unfold the Second Section.

STEP 3. Open the Ramp.

STEP 4. When you are satisfied that the ramp is positioned properly, drill a $\frac{1}{4}$ " hole $1\frac{1}{2}$ " deep into the landing or vehicle floor using the prepunched holes in the top adapter lip as a guide. Secure with the two supplied Pins.

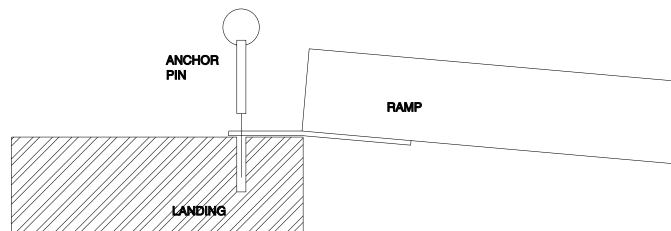

Maintenance & Safety

1. Regularly check your ramp for worn, loose, or damaged parts. If you find any of these conditions **DO NOT USE THE RAMP**. Contact your Dealer for repairs. Failure to do so may cause severe injury !
2. The surface of the ramp may become slippery in icy or wet conditions. Caution should be used when using the ramp.
3. Keep the ramp surface free of dirt and debris.
4. Always check the chair or scooter for the manufacturer's recommended degree of incline. Do not use ramps that are steeper than recommended.
5. Do not add to or modify any part of the ramp. Doing so may void your warranty.
6. Do not use the ramp for purposes other than those it was designed for.
7. Clean ramps with water and mild detergent.
8. Use caution when folding and unfolding your ramp. Beware of pinch points.
9. Be sure that the ramp is always on a solid landing and firmly placed. Always anchor your ramp in place with the pins provided.

bridging the gap.

ALUMIRAMP, Inc. 855 Chicago Road, Quincy, MI 49082 tel 800.800.3864 fax 800.753.7267